Cohesive Bandage

Elastic cohesive fixation bandage to be used on intact skin.

Tear by Hand Easily

Anti-Slip

Non-Adhesive

Water Repellent

Breathable

Soft Touch

Intended Use

Ories Cohesive Bandage are intended for use to secure and prevent movement of primary and secondary dressings, splints, casts, padding materials, catheters and cannulas.

Product Description

- ▶ Ories Cohesive Bandage is a cohesive elastic wrap constructed from 100% polyester made by a special weaving method.
- ▶ Cohesive bandage adheres to itself, not to skin, hair or clothing. There is no need to another tap to secure. The layers of the bandages do not slip so it can obtain optimal secure.
- ▶ Not made with natural rubber latex.
- ▶ It is supplied non-sterile.


Patient Target Groups

The patient who needs to secure and prevent movement of primary and secondary dressings, splints, casts, padding materials, catheters, and cannulas.

Warnings


- ▶ DO NOT place FOXXMED Cohesive Bandage in direct contact with an open wound because it may lead to infection.
- ▶ DO NOT tighten the bandage excessively when pasting, so as not to cause skin damage.
- ▶ DO NOT reuse this product. Reuse may result in compromising product integrity or cross infection.
- ▶ This product is intended to be used for a maximum of 24 hours.
- ▶ Any serious incident that occurs in relation to the device shall be reported to the manufacturer and the competent authority of the EU Member State in which the user and /or patient is established.

Instructions for Use **Note:** Tear by hands to achieve optimal performance and stability.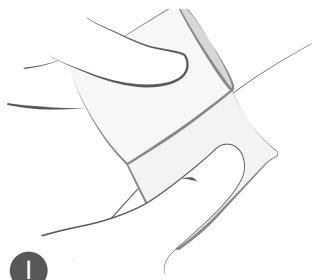


1

Keep the bandage close to the specified position. Apply one and half to two circle and apply tension to obtain desired support and compression.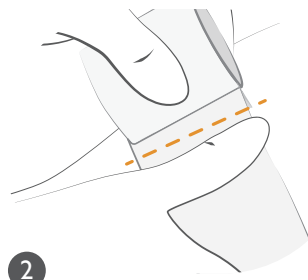


2

Hold the bandage with one hand, and put the thumb of the other hand on the area intended to be torn.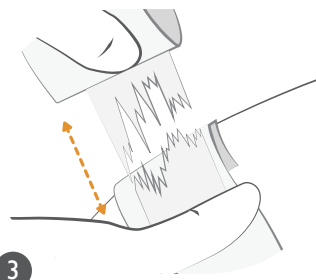

3

Use the thumb slightly press the bandage, and another hand which hold the bandage tear in parallel direction.

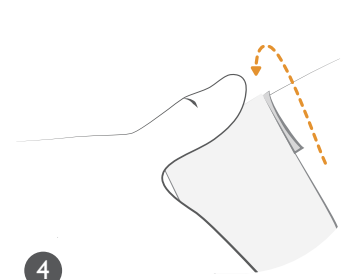4

Gently press end of bandage to secure the terminal end of the bandage.

FOXXMED LTD., No. 60, Jianjin 2nd Rd., Dongshan Township, Yilan County 269, Taiwan (R.O.C.)

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